
# OXIMETER X3

## Pulse oximeter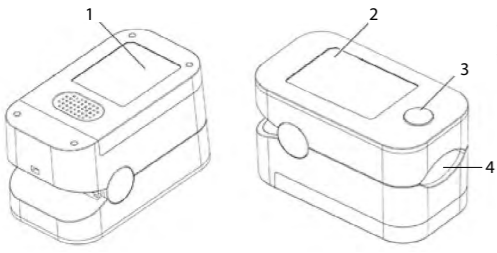

### EN – USER MANUAL

#### 1. BOX CONTENTS

- 1 x TrueLife Oximeter X3
- 2 x AAA alkaline battery
- 1 x Suspension cable
- 1 x Case
- 1 x Manual
- 1 x Safety instructions


The TrueLife Oximeter X3 uses a light source that passes through the tissue and hits a detector that measures its intensity. The Pulse Oximeter X3 measures oxygen saturation or blood oxygen levels as well as heart rate.

#### 2. PRODUCT DESCRIPTION

- Battery cover
- Display
- Control button
- Finger clamp

#### Display Description

- Oxygen saturation symbol
- Oxygen saturation (percentage value)
- Pulse rate waveform
- Battery status symbol / Bluetooth symbol (alternately)
- Heart rate symbol
- Perfusion index
- Pulse rate
- Pulse rate bar

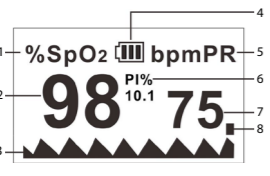
#### 3. BATTERY INSTALLATION


- Remove the battery cover on the back of the oximeter and insert 2 AAA batteries. Make sure that the batteries are inserted with the correct polarity as indicated in the battery compartment. Do not use batteries after their expiration date.
- If the battery status symbol flashes on the display, the batteries must be replaced. When battery power is critically low, the device will not take a reading.
- If you will not be using the device for an extended period of time, remove the batteries.

#### 4. SETTINGS

- The oximeter offers the possibility to set audible alarms, pulse rate beep and brightness. You can then set the upper and lower limits for the audible alarms.
- To enter settings, turn on the oximeter by pressing the control button. After the display turns on, hold down the control button again for 2 seconds. The display shows the menu for setting the alarms, the pulse rate beep and the brightness. The menu for setting the alarm limits is accessed by holding down the control button while the asterisk is next to „Alm setup“.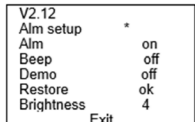

Setting the sound indicator, heartbeat soundtrack and brightness

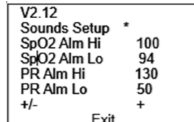Setting the boundaries of the sound indicator

- To exit settings, navigate to the line „Exit“ and hold down the control button .

#### Setting the measurement alert

- The reading can be accompanied by an audible alert.
- To turn on the alert, scroll to the „Alm“ line and hold down the control button until „on“ is set. To turn off the alert, proceed in the same way and set to „off“.

#### Setting audible alarm limits

- When taking a reading, the oximeter starts emitting an audible alarm when one of the measured values is smaller or larger than the set limits.
- To change the limits, enter the alarm limit menu and select the line with the value you want to change. Once you are at your desired value, hold down the control button and the value will change. If „+“ is on the bottom line, the value will rise. If „-“ is there, the value will decrease.

**Note: We recommend that you leave the alarm limits at the preset values.**

#### Setting the heart rate beep

- The pulse rate can be accompanied by a beep when a reading is being taken.
- To turn on the pulse rate beep, navigate to the „Beep“ line and hold down the control button to change it to „on“. To switch off, proceed in the same way and set to „off“.

#### Adjusting the brightness

- The brightness of the oximeter display can be adjusted.
- To change the brightness, navigate to the „Brightness“ line and hold down the control button to set the desired brightness, where „1“ is the lowest and „5“ the highest.

#### Factory settings

- If you want to restore factory settings, navigate to the „Restore“ line and hold down the control key „OK“ will appear on the line and all lines will be returned to their original values.

#### 5. TAKING A READING

The measurement process itself is very simple. Follow the instructions below.

- Insert your finger into the finger clamp. Keep your finger still.
- Press the control button. The oximeter immediately starts measuring. Do not move during the measurement process.
- After a few seconds, the reading appears on the display.
- There are 6 different display modes on the device. Use the control button to toggle between them.
- After removing your finger from the clamp, the oximeter will automatically shut off after 10 seconds.

#### 6. READINGS

| SpO2 (oxygen saturation) in% | Classification / Necessary measures      |
|------------------------------|--|
| 99-94                        | Normal values                            |
| 94-90                        | Lower values: doctor's visit recommended |
| <90                          | Critical values: doctor's visit needed   |

**Note: The table does not apply to persons with certain diseases - see Safety instructions (e.g. asthma, heart failure, respiratory diseases) and when staying at altitudes above 1,500 m. If you suffer from any of the diseases mentioned, always consult your doctor, who will assess the readings.**

#### 7. SPECIFICATIONS

|                              |  |
|------------------------------|--|
| <b>Measurement range</b>     | Oxygen saturation: 1–100%<br>Heart rate: 0–250 beats per minute                  |
| <b>Measurement deviation</b> | Oxygen saturation: 70–100%, ± 2%<br>Heart rate: 30–250 bpm, ± 2 beats per minute |
| <b>Measurement sensor</b>    | Red light (wavelength 660 nm); infrared light (wavelength 905 nm); silicon diode |
| <b>Battery</b>               | 2x 1.5V AAA  |
| <b>Bluetooth</b>             | Yes - 4.0  |
| <b>Operating conditions</b>  | Temperature: 5 °C–40 °C (41 °F–104 °F), Humidity: ≤ 95% RH                       |
| <b>Storage conditions</b>    | Temperature: -25 °C–55 °C (-13 °F–131 °F), Humidity: ≤ 95% RH                    |
| <b>Dimensions</b>            | 62 x 37 x 32 mm  |
| <b>Weight</b>                | 42.5 g (without batteries)   |

#### SAFETY WHEN WORKING WITH THE DEVICE:

- Read the attached safety instructions carefully.
- For proper handling of the product, read the latest version of the manual, which is available for download at: [www.truelife.eu](http://www.truelife.eu).

Types and information changes reserved.

**Warning: This product is intended for use in households and similar environments. We declare that if the TrueLife Oximeter X3 product is used as a professional medical device, a safety inspection (BTK) must be performed regularly at the following intervals: 24 months after the purchase of the product, then every 12 months.**

#### Importer:

elem6 s.r.o.,  
Braškovská 15, 16100 Prague 6,  
Czech Republic

Made in P.R.C.

MedPath GmbH  
Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany

Shenzhen Jumper Medical Equipment Co., Ltd.  
Address: D Building, No. 71, Xintian Road, Fuyong Street, Baoan, Shenzhen, Guangdong, China, 518103

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